



## QUICK GUIDE

### Unfolding the eFOLDi Lite Scooter

eFOLDi 


Before starting stand the scooter upright in an open space away from pets and small children.

Unfolding the scooter is straightforward and with a little practice you will be able to set it up in seconds.

We advise you to practise with the scooter before venturing out for the first time.



**1** Release the yellow steering column latch...

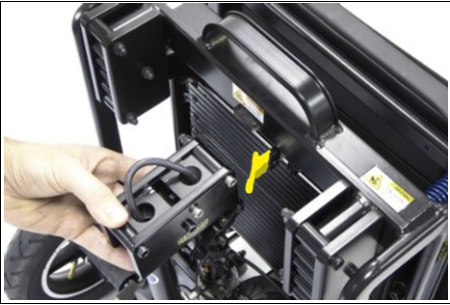

**2** ...and allow the column to swing free.



**3** Pull the handlebars out and upward..



**4** ...until they lock into a vertical line.



**5** Locate small yellow toggle on side and pull to release front wheel assembly.



**6** Hold the steering column and seat back and pull them apart...



**7** ...the scooter will unfold as you pull the seat back and steering column apart and ...



**8** ...the scooter will lock itself into its open position.



**9** Make sure the locking latches have locked to the steel bar...



**10** ...if not press your foot down onto the footplate to lock the scooter open.



**11** If not already installed apply the seat cushions and seat back as shown.






**12** Check that the magnetic brake is in the LOCKED position to engage the motor.

## Folding the eFOLDi Lite Scooter

		
<p><b>1</b> Fold the seat backrest down to the seat base.</p>	<p><b>2</b> If the scooter has armrests pull the black toggle to release then fold backrest down as per (1)</p>	<p><b>3</b> Press and hold the small yellow latch on the steering column...</p>
		
<p><b>4</b> ...and fold the column down until it is locked by the magnetic catch.</p>	<p><b>5</b> Firmly hold the scooter by its steering column and the carry handle and firmly press the column down towards the footplate</p>	<p><b>6</b> Tip the scooter back at an angle and use your foot behind the wheel to stop it moving away from you</p>
		
<p><b>7</b> ...press inwards on the steering column until the front wheel locks into place...</p>	<p><b>8</b> ...and the steering column latch locks into position.</p>	<p><b>9</b> Press in steering column to lock it tightly against the frame.</p>

## eFOLDi Lite Operation

		
<p>To turn the scooter press the grey button below the battery display.</p> <p>Select (D) Drive or (R) Reverse and press the thumb throttle to go.</p>	<p>The battery can be charged using the charging port as shown.</p> <p>Always recharge the battery as soon as possible when the display is indicating amber.</p>	<p>The battery can be removed by lifting the latch and lifting the battery upwards.</p> <p>After removal the battery can be charged using the socket on the side of the battery.</p>